

# CLINITEST Rapid COVID-19 Antigen Self-Test

## Quick Reference Instructions

The US Food and Drug Administration has granted a shelf-life extension for the CLINITEST® Rapid COVID-19 Antigen Self-Test from 11 months to 24 months from the date of manufacture.


This test is free and distributed by the U.S. Health Resources & Services Administration (HRSA) through NEMS to ensure and expand free and accessible testing to the community. It is not authorized for re-sale.

\*This test is authorized for self-testing use by individuals aged 12 and older (or adult-collected samples from individuals ages 2-13).

Please scan the QR CODE for updated information.




### STEP 1

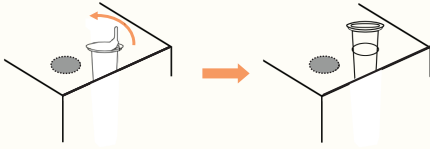

Before starting, wash your hands. Place tube in the holder and remove the seal. Make sure the tube is standing straight.

### STEP 2

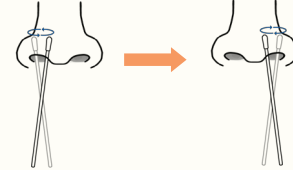


5 circles

5 circles

Carefully insert swab tip about 1/2 to 3/4 inch deep into nostril and complete at least 5 circles in **EACH** nostril.


### STEP 3

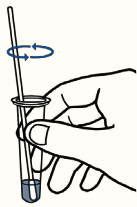

6 circles

Insert swab tip into the tube and mix vigorously at least 6 times on the bottom and side of the tube.

### STEP 4

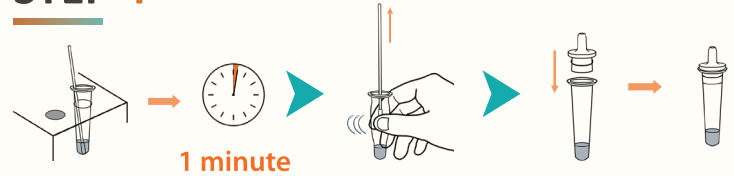

1 minute

Place tube in the tube holder. Start timer for 1 minute. After 1 minute, squeeze swab tip several times and remove from the tube. Insert tube tip into the top of the tube.

### STEP 5



Result window

Sample well

15 minutes

Squeeze 4 drops of solution into the sample well and start timer for 15 minutes.

### STEP 6 READ TEST RESULT AFTER 15 MINUTES



**COVID-19 POSITIVE:** Two pink/red lines indicate a positive result. Immediately notify your health care provider about your positive result to discuss COVID-19 treatment options. Note that oral antiviral treatment for COVID-19 must be started within 5 days of symptom onset.



**COVID-19 NEGATIVE:** COVID-19 viral proteins were not detected. Consider retesting in 1-2 days if your symptoms started within 3 days ago.



**INVALID:** This test result is invalid. Perform a new test.



**NORTH EAST  
MEDICAL SERVICES**

東北醫療中心

a california *health+* center

www.nems.org  
NEMS - Rev. 01/27/2023